

ACU-PAC PREGNANCY TEST KIT

High Sensitivity 20 IU/L




- Rapid determination of human chorionic gonadotropin (hCG) in urine specimens. This test kit is used to obtain a visual, qualitative result for early detection of pregnancy.

STANDARDIZATION


> The Acu Pac Pregnancy Test has been standardized to World International Reference Preparation (WHO 1st IRP 5/537)

INTERPRETATION OF RESULTS

Negative : Only one colour band appears on the Control (C) region. No apparent band on the Test (T) region.

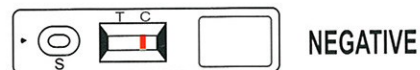

Positive : Distinct colour bands appear on the Control (C) and Test (T) region. Colour intensity of the bands may vary.

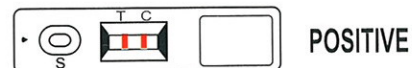

Invalid : No visible band at all or no visible band at the Control (C) region. Repeat test.



Warning and Precaution

- For in vitro diagnostic use only
- Do not use kit beyond expiry date.

Distributed By:

ACU-PAC PHARMA (M) SDN. BHD. (Perak, Kelantan, Terengganu)

SCANLAB SDN. BHD. (Northern, Southern States)